Nu Radiance®
Classic
teeth whitening system

Classic teeth whitening system

PATIENT INSTRUCTIONS

Your dentist has provided you with one of the most advanced teeth whitening systems available for custom tray whitening. With unsurpassed results, Nu Radiance Classic contains a pleasant-tasting whitening gel that is available in multiple strengths to accommodate your teeth whitening needs.

Each syringe provides enough gel for about 6 complete applications for upper and lower teeth. Each segment marked on the syringe is the approximate dose for whitening two arches.

To whiten your teeth:

1. Snap off and save the small cap from the Classic teeth whitening syringe tip.
2. Express a small dot (● actual size) into the deepest portion of each tooth compartment of the custom custom whitening trays. Place gel only into the tooth compartments that are in your smile (6 to 8 teeth across). Excess gel in the tray can lead to temporary irritation and discoloration of gum tissue.
3. Blot your teeth dry with a tissue.
4. Position the custom whitening trays containing whitening gel onto the teeth. Press carefully. Pressing too hard will push the whitening gel onto your gums and soft tissue. Avoid direct contact of the Classic whitening gel with gums or soft tissue.
5. Remove excess Classic whitening gel with a cotton swab, tissue, or toothbrush.
6. After inserting upper and lower trays, place the small cap on the Classic syringe tip for storage.
7. Treatment times are:

10% carbamide peroxide	6-8 hour wear-time*
16% carbamide peroxide	2-4 hour wear-time*
22% carbamide peroxide	1 hour wear-time*
30% carbamide peroxide	30 minute wear-time
Once a day or as directed by your dentist	

**Can be worn overnight*

8. Once treatment is completed, remove custom whitening trays. Brush teeth and custom whitening trays with toothpaste. Rinse well. Rinse custom whitening trays with room temperature water. Warm or hot water can distort and ruin custom whitening trays.
9. Place custom whitening trays into storage case.

Whitening will only improve your natural teeth. It will not change the shade of crowns, bridges, veneers, composite fillings, dentures or any other man-made material. If you are considering dental work, it is best to whiten your natural teeth first, and then your dentist can select the whiter shade for your restorative dentistry.

Results may vary

Final results may vary based on wear-time and frequency of applications. For best results, follow the treatments that your dentist recommends, the instructions listed in this pamphlet, and the "Tips for better whitening" section below.

Tips for better whitening

1. The best whitening results are achieved when patient maintains regular dental hygiene visits.
2. Do not smoke, eat or drink while wearing custom whitening trays.
3. Avoid the use of tobacco and food/beverages that are deep in color such as coffee, tea, red wine, and colas during and up to 24 hours after a whitening treatment.

Possible side effects

Patients have reported sensitivity to cold during teeth whitening treatment and occasional gum irritation. A smaller number of patients have reported temporary discomfort during treatment such as tongue and/or lip soreness, throat irritation or moderate tooth pain. Sensitivity and tissue soreness usually subside within 1 to 3 days after treatment.

If you experience sensitivity or tissue irritation, you may skip whitening treatments until sensitivity subsides.

A dark or purple color may be seen in the dental trays after treatment. This occurs when whitening gel oxidizes the surface stains on amalgam (silver) fillings. This is normal and will not harm you, your amalgam filling, or your dental trays.

Store at temperatures not exceeding 73° F (23° C), protect from direct light, especially sunlight and heat sources. Refrigerate for long-term storage to maintain product stability. Do not freeze.

Keep out of the reach of children.

WARNING: Whitening is not recommended for children under 14 years old. Throat irritation and nausea may be experienced if excessive whitening gel is swallowed. Do not use if you are pregnant or breastfeeding. Do not use if you are allergic to any of the ingredients listed below:

Active ingredient: carbamide peroxide.

Other ingredients: glycerin and/or propylene glycol, buffered polyacrylic acid and peppermint oil.

Nu Radiance® Classic

teeth whitening system

Your Dentist's Recommendations:

Pre-Whitening Shade:

Post-Whitening Shade:

Follow-Up Appointment:

Questions? Call your dental office at:

Nu Radiance products are only available through your dentist.

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